


For a 3D view of any instrument tip, visit our online catalog at www.am-eagle.com

WHAT ARE DOUBLE GRACEY™ INSTRUMENTS?


Truly a double Gracey design, these instruments combine the best features of Gracey and Universal curettes. Double Gracey instruments have rounded toes and two (2) cutting edges. Advantages of using Double Graces include easy adaptation to mesial and distal surfaces, as well as buccal and lingual surfaces, without switching or repositioning the instrument. The Mini versions allow easier access into deeper pockets, tighter interproximal areas, and furcations. All Double Gracey instruments are sharpen-free with American Eagle Instruments' patented XPSM Technology.

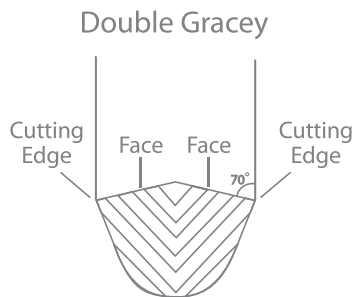

*All Double Gracey curettes replace Universal curettes.


The face of a Double Gracey is raised, thus creating two slightly sloped faces, approximately 110 degrees, measured from the terminal (lower) shank. The face and blade angles allow the practitioner to use Double Gracey instruments exactly how they would use traditional Gracey curettes—the terminal shank parallel to the tooth axis while performing the cleaning.

- Anterior
- Posterior
- Mini Anterior
- Mini Posterior

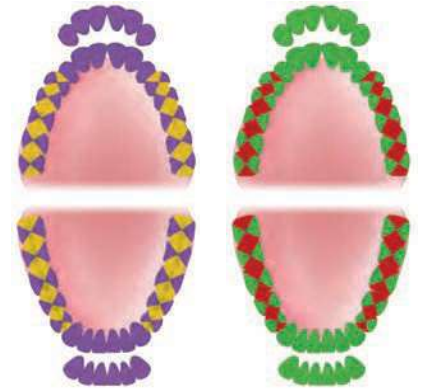

DOUBLE GRACEY ANTERIOR & MINI ANTERIOR

- Enables clinician to scale from light to heavy calculus
- Maneuver between buccal and lingual surfaces without switching position
- XP Technology creates less sensitivity on cementum

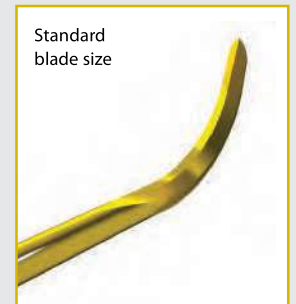

Standard blade size

DOUBLE GRACEY POSTERIOR & MINI POSTERIOR

- Enables clinician to remain in a 12 o'clock ergonomic position for the duration of cleaning
- Allows clinician to keep lower shank parallel to tooth surface
- Maneuver between mesial and distal surfaces without switching position



Mini blade size

